Disposable Single Patient Use Repositioning Sling

- The Disposable Repositioning Sling is a full bed-length sling that allows the patient or resident to remain in a supine position for transfer or in-bed repositioning. It can also be used for turning.
- Available in standard and bariatric sizes.
- Made of breathable polyester with polyester straps
- **Always ensure you have read the Sling Application Guide before use**

Code	Size	Safe Working Load (SWL)
TO17LX-PSS	REGULAR	600 lbs / 272 kg
TO17LXB-PSS	BARI	1000 lbs / 454 kg

General Information

- High quality product with many safety features within the material design and content of the product
- Size/colour coordinated on the binding, handles and displacement straps
- Can be used for bathing, showers and pools. Dispose of sling after bathing, etc, and replace with a new sling
- Use with confidence - the slings are not going to rip or tear or fall apart!
- Disposable Slings can also be left under the patient
- Disposable Slings are intended to be used as a patient specific product
- Dispose of sling after use.

Wash Indicator

Disposable Repositioning Slings incorporate a wash indicator label. In the event of the sling being washed (and then recycled for use again) the wash indicator will turn red so the care provider is aware that the sling has been washed.

Do not re-use. Dispose sling after use.

Prior to Washing

Wash Indicator

The wash indicator will highlight that the sling has been washed.

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**ALWAYS INSPECT SLING
BEFORE EACH USE**

WARNING

IMPROPER sling size, improper sling attachment or improper sling and lift inspection can cause death or serious injury. Read instructions and warnings in manual.

LOCK bed and wheels before lifting. Keep side rails up.

ENSURE all sling straps are properly attached to lift spreader bar.

DO NOT exceed weight capacity of sling and lift.

Visibly inspect sling prior to each use to ensure sling is not damaged or wasted.

NO portion of patient should fall outside of sling.

HANDLES on sides do not support weight and are for convenience.

Lift resident 1-2 inches over bed or chair to ensure sling is holding resident securely before further lifting.

NEVER leave a patient unattended in a sling.

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Sling Application Guide

Patient Turning (Using Repositioning Sling)

- 1 Ensure spreader bar is PERPENDICULAR to patients shoulders and is located just above the abdomen and attach 2 lower sets of loops to the center hook of spreader bar as shown (use red loops).

- 2 Attach 2 upper sets of loops to the center hook of spreader bar as shown (use red loops).

- 3 Press UP button on hand controller to roll patient.

Supine Lifting (Using Repositioning Sling)

- 1 Apply repositioning sling by log rolling patient

- 2 Ensure spreader bar is PERPENDICULAR to patient's shoulders and located just above the abdomen

- 3 On each side of patient, attach two lower sets of loops to spreader bar as shown (use green loops)

- 4 On each side of patient, attach two upper sets of loops to spreader bar as shown (use blue loops)

- 5 Attach set of head loops if needed (red). Lift patient.

Please use blue loops to adjust patient/resident positioning when necessary

READ WARNINGS ON OPPOSITE SIDE BEFORE USE